

### Drug testing solutions

DrugWipe is the leading choice for law enforcement, border control, rehabilitation clinics and prison systems worldwide.

#### *Road safety and drug driving*

Suspensions of drug abuse can be quickly and reliably established at roadside checks.

#### *Rapid drug test*

Easy, quick and hygienic handling. One of the best drug tests in the world with lowest sampling volume and easy to read color indicator.

ACS is the exclusive distributor of DrugWipe in North America



#### Alcohol Countermeasure Systems

60 International Boulevard  
Toronto, Ontario M9W 6J2 CANADA

+1 416 619 3500

[acs-corp.com](http://acs-corp.com)  
[drugwipe.ca](http://drugwipe.ca)

ACS, ALCOHOL COUNTERMEASURE SYSTEMS and the "Molly" are trademarks of Alcohol Countermeasure Systems (International) Inc. and are used under license.

DrugWipe and WipeAlyser are a trademark of Securetec Detektions-Systeme AG and used under licence.

# DrugWipe® A

## Universal Drug Screener



20181002



ALCOHOL COUNTERMEASURE SYSTEMS

# Reliable, hygienic drug detection in minutes

## Surface, sweat, saliva drug test

DrugWipe A detects relevant substances in saliva, in sweat (on the skin) and on surfaces, indicating that the test subject has recently consumed or handled controlled substances.



## How does it work?

The test starts as soon as you break the integrated glass and release the liquid. The liquid helps the drug particles that are bound to the antibodies on the test strip migrate to the test line. A sequence of red lines appear on the read out window, depending on the results.

## Drugs detected

- Cannabis
- Amphetamines
- Methamphetamines
- Cocaine
- Opiates

## DrugWipe A advantages

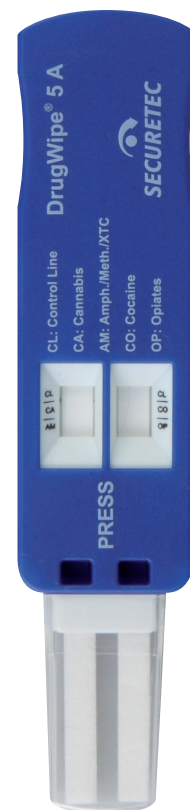
- Test is quick and easy to administer
- Analysis and results in 3 to 8 minutes
- Certainty in detection of controlled substances

DrugWipe A provides accurate results with distinctive red test lines that appear even for slight traces of drugs. High specificity and reliability (over 95%) as confirmed by laboratory studies.

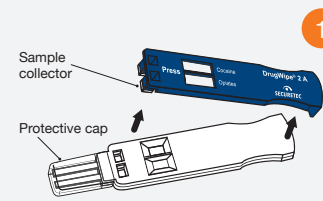
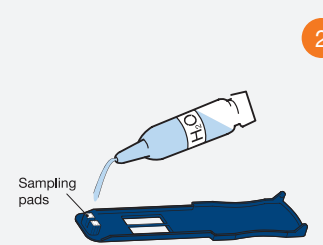
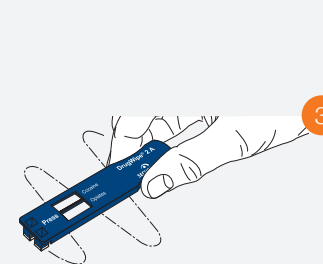
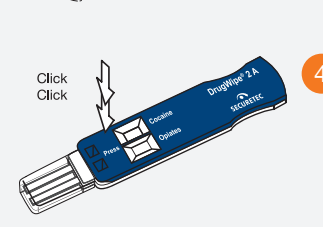
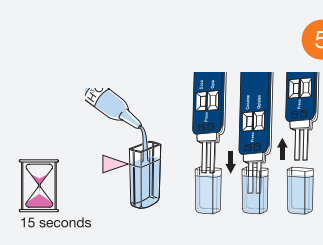
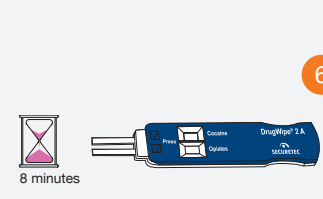
## Quality control

DrugWipe A contains a quality control line (CL) that authenticates functionality, all control lines must turn red after the test to ensure validity

DrugWipe A is an immunological rapid screening test. The sample collector transfers the sample to the test strips, which contain drug-specific antibodies. If the sample contains drugs, they will bind to the antibodies. The test begins as soon as the test strip is immersed in water.



## Test instructions

- 1 Remove the blue sample collector from the white test cassette.  
*Do not remove the protective cap. Do not touch the sampling pads.*
- 2 **For skin/sweat/surfaces.** Moisten the sampling pads on the sample collector. Shake off any excess water.  
**For saliva screening.** Do not moisten the sampling pads, as saliva is collected with a dry sample collector.
- 3 Using light pressure, wipe the surface (e.g. tongue, forehead, steering wheel) with the sample collector approx. 5 to 6 times. Only dab the surface if it is very rough.
- 4 Place the sample collector back on the test cassette. *The sample collector must lock in place with an audible double click.*
- 5 Remove the protective cap and fill with water up to the marked level. Immerse the test strip in water for 15 seconds and then remove. *Do not immerse the white plastic casing when doing so.*
- 6 *Leave the test undisturbed on a horizontal surface.* After 8 minutes, read the test result.