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© 2015 - 2016 Alcohol Countermeasure Systems

ACS, ALCOHOL COUNTERMEASURE SYSTEMS, DRIVESAFE and the "**Molly**" are trademarks of Alcohol Countermeasure Systems (International) Inc. and are used under license. Alcohol Countermeasure Systems is the trading style of Alcohol Countermeasure Systems (International) Inc.

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DRIVESAFE™ elite
BREATH ALCOHOL TESTER

ACS
ALCOHOL COUNTERMEASURE SYSTEMS

QG-MULTI-60-000718-C
© 2015 - 2016

Quick guide

FEDERAL COMMUNICATIONS
COMMISSION (FCC) DISCLAIMER

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with Part 15 of the FCC rules. Operations are subject to the following 2 conditions:

- 1. This device may not cause harmful interference.
- 2. This device must accept any interference received, including interference that may cause undesired operations.

INDUSTRY CANADA

This device complies with Industry Canada licence-exempt RSS standard(s). Operation is subject to the following two conditions:

- 1. This device may not cause interference, and
- 2. This device must accept any interference, including interference that may cause undesired operation of the device.

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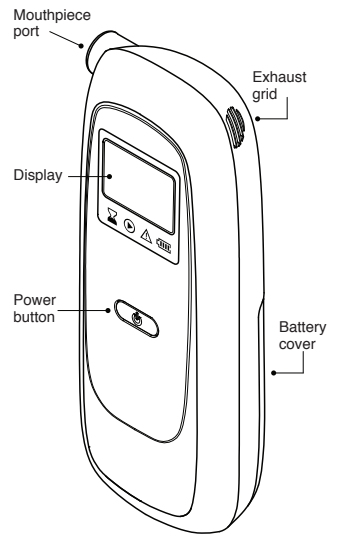
Alcohol Countermeasure Systems Corp
60 International Boulevard
Toronto, Ontario M9W 6J2 CANADA
info@acs-corp.com
acs-corp.com

ENG
DRIVESAFE elite DIAGRAM

Thank you for purchasing the DRIVESAFE elite person breath alcohol tester, a device designed for your safety.

This tester measures Breath Alcohol Concentration (BrAC). In some cases, it may take up to 60 minutes to reach the highest BrAC after the last consumption of alcohol.

Each person reacts uniquely to the consumption of alcohol and may become impaired at a point below the legal limit.

SAFETY AND PRECAUTIONS

- Use this tester for its intended purpose only
- Do not blow smoke or spit into the mouthpiece; it may damage the sensor
- Always use a new mouthpiece when blowing into the tester
- Do not cover the exhaust grid (the top-right side of the tester) while providing a breath sample
- Do not store the tester in extreme heat or cold and avoid exposing it to direct sunlight for extended periods of time
- Use the tester within the ambient temperature range of 5 to 50 °C
- Only use AA alkaline batteries
- Turn the tester off before removing the batteries
- Keep magnets or radio frequency (RF) devices away from the tester to minimize the potential for electromagnetic interference
- Wait 10 minutes after drinking, eating or smoking before providing a breath sample
- Drinking alcohol and eating foods containing alcohol will result in a warning, including: Medicine, menthol drops, smoking, using mouthwash or mouth spray
- Do not spray air fresheners, disinfectants, cologne or perfume prior to a test, as these may also contain alcohol

	Wait	The tester is warming up or analyzing a sample.
	Ready	The tester is ready to accept a breath sample.
	Error	An error has occurred. Wait a few seconds to try the test again.
	Battery strength	Indicates that the batteries need to be replaced.

TESTING IN NORMAL CONDITIONS

1. Press the power button to turn the tester on.
2. Wait until the cursor is above the **Ready** icon.
3. Blow into the mouthpiece.
A continuous tone can be heard, indicating proper breath flow. Continue blowing until the tone ends and the tester clicks. A failure tone with the **cursor above the Error icon** indicates that a proper breath sample was not provided. Wait until the cursor is above the **Ready** icon to try the test again.

The result will be displayed after a few seconds and indicated by an audible tone.

Result	Tone
Pass	Continuous
Warn	Slow, intermittent
Fail	Fast, intermittent

TESTING IN A COLD ENVIRONMENT

When blowing repeatedly into the tester within a cold environment (below 10 °C), condensation droplets may form inside the blow channel.

When providing a breath sample, tilt the tester as shown:

After several breath tests have been performed, blow quickly into the mouthpiece to clear any condensation.

NOTE: Always use a new mouthpiece when blowing into the tester.

BATTERY POWER

The DRIVESAFE elite tester requires 2 AA alkaline batteries for operation.

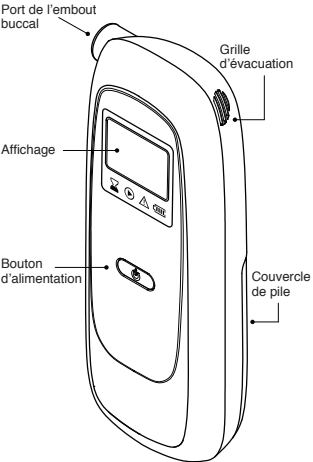
In production, a plastic tab is inserted between the battery and the electrical connection to conserve battery power during shipment and storage.

To operate the tester for the first time, pull the plastic tab from the battery compartment, which is accessible at the back of the tester.

DISCLAIMER

Sale of the DRIVESAFE elite personal alcohol tester is on the express condition that neither the manufacturer nor the vendor of the device shall be held responsible for the accuracy of any breath alcohol test reading obtained from the device, nor liable for any loss, damage, injury or other consequence resulting, directly or indirectly, from the use thereof.

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