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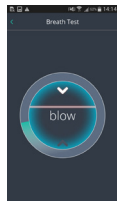
Site Web : acs-corp.com

© 2015 Alcohol Countermeasure Systems

Alcohol Countermeasure Systems Corp
60 International Boulevard
Toronto, Ontario M9W 6J2 CANADA
info@acs-corp.com
acs-corp.com

ACS, ALCOHOL COUNTERMEASURE SYSTEMS, DRIVESAFE and the **"Molly"** are trademarks of Alcohol Countermeasure Systems (International) Inc. and are used under license. Alcohol Countermeasure Systems is the trading style of Alcohol Countermeasure Systems (International) Inc.

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DRIVESAFE™ elan
MOBILE BREATH ALCOHOL TESTER

ACS
ALCOHOL COUNTERMEASURE SYSTEMS

QG-MUL-60-000547-B
© 2015

Quick guide

FEDERAL COMMUNICATIONS
COMMISSION (FCC) DISCLAIMER

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with Part 15 of the FCC rules. Operations are subject to the following 2 conditions:

- 1. This device may not cause harmful interference.
- 2. This device must accept any interference received, including interference that may cause undesired operations.

INDUSTRY CANADA

This device complies with Industry Canada licence-exempt RSS standard(s). Operation is subject to the following two conditions:

- 1. This device may not cause interference, and
- 2. This device must accept any interference, including interference that may cause undesired operation of the device.

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ENG

DISCLAIMER

Sale of the DRIVESAFE™ elan mobile breath alcohol tester is on the express condition that neither the manufacturer nor the vendor of the device shall be held responsible for the accuracy of any breath alcohol test reading obtained from the device, nor liable for any loss, damage, injury or other consequence resulting, directly or indirectly, from the use thereof.

NOTE: This product works with all recent smartphones running on the Android platform. For older smartphones, please verify that the USB HOST mode is supported.

SAFETY AND PRECAUTIONS

- Use the DRIVESAFE elan tester for its intended purpose only
- Do not blow smoke or spit into the mouthpiece; it may damage the sensor
- Always use a new mouthpiece when blowing into the tester
- Do not cover the exhaust grid (the top-left side of the tester) while providing a breath sample
- Do not store the tester in extreme heat or cold and avoid exposing it to direct sunlight for extended periods of time
- Use this tester within the ambient temperature range of 5 to 50 °C
- Keep magnets or radio frequency (RF) devices away from the tester to minimize the potential for electromagnetic interference
- Wait 10 minutes after drinking, eating or smoking before providing a breath sample
- Do not spray air fresheners, disinfectants, cologne or perfume prior to a test, as these may contain alcohol

MOBILE APP

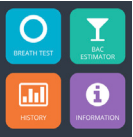
In order to use the DRIVESAFE elan, you must first download the elan app for your smartphone. Once downloaded, open the app and pair it with the elan mobile tester via the mini USB cable provided.

SETUP PROCESS

Follow these steps to properly pair a smartphone with the elan mobile tester:

- 1. Download the elan app from Google Play.
- 2. Start the app. Attach one end of the mini USB cable into the cable port of the tester and the other into the cable port of the smartphone. The tester will power on automatically.
- 3. A registration form is displayed. Fill out the form in its entirety and tap **Activate**.


NOTE: The Serial # field is filled out automatically when connecting the tester to a smartphone. The tester can only be registered once. Once activated, it is not possible to change these details.

Breath Test	To take a breath test.
BAC Estimator	To estimate BAC without using the tester.
History	To view and compare past test results.
Information	General information about BAC and BrAC.

BREATH TEST

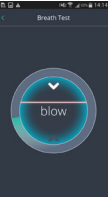- 1. Tap the **BREATH TEST** icon on the home screen. Wait until the app initializes and displays **ready**.

The app displays the results after a few seconds. You have the option to **Tweet my Result, Call a friend for a ride or Call Taxi**.

BAC ESTIMATOR

NOTE: Use of this app is on the express condition that neither the manufacturer nor the vendor of the device shall be held responsible for the accuracy of this approximation, nor liable for any loss, damage, injury or other consequence resulting, directly or indirectly, from the use thereof.

The BAC Estimator allows you to roughly estimate your alcohol level without the use of the tester.

- 1. Tap the **BAC ESTIMATOR** icon on the home screen. A bar graph is displayed.
- 2. Tap the gear icon at the top of the screen to open the **Personal Details** form. Fill out the form in its entirety by inputting your age, height, weight and gender. Tap **Next** to apply these settings.
- 3. Tap **Add Drink** at the top of the screen. A drink form is displayed. Input the volume, percentage and the time when it was consumed. When completed, tap **Add Drink** at the bottom of the screen. Repeat this step for each alcoholic drink consumed.

Inputted drinks will appear at the bottom whereas the bar graph shows the estimated alcohol level. You can review previous estimations by scrolling through the graph or by selecting a different date.

HISTORY

The history function allows you to see and compare different breath test results recorded with either the tester or the BAC Estimator.

- 1. Tap the **HISTORY** icon on the home screen. A bar graph displays previous breath test results recorded with the tester. Scroll up and down to see detailed time stamps for each test taken.

- 2. Tap **Compare** at the top of the screen. A bar graph with comparable results is displayed. Orange icons represent test results recorded with the BAC Estimator and blue icons represent test results recorded with the tester.

Tap the check mark beside either **BAC Estimator** or **Actual Breath Test** to add or remove them from the bar graph.

WARRANTY

The DRIVESAFE elan mobile breath alcohol tester is warranted against defects in materials or workmanship for 1 year from the date of purchase. During the warranty period, Alcohol Countermeasure Systems agrees to replace or repair any unit or part that in its judgement is found to be defective, providing the defect is not due to damage or misuse.

SERVICE

Testers requiring service should be packed securely and shipped prepaid to an authorized Service Centre. Contact Alcohol Countermeasure Systems for more details:

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